

ENGLISH**Sterile packed, single use orthopaedic instrumentation****Specific product information**

For detailed information concerning individual packed devices (such as product code, description, lot number, shelf life, single use designation, and method of sterilization) please refer to the labelling on the package. Labelling specifies the connection properties of the device as either *Quick Fit* or *Plain Shank*.

Indication

Preparing bones for insertion of implants in orthopaedic surgery.

Use

Ensure that you are familiar with the recommend uses, compatibility and correct handling of the device. Follow all standard operation procedures, including irrigation of the working field, to minimise the risk of injury to the patient. Only use the device for its intended purpose. Do not modify the design in any way.

Warnings

This device is labelled as single use and must be discarded at the end of a surgical procedure. It must not be re-used even if it appears to be undamaged. This product may contain nickel. A nickel allergy test is recommended if there is any doubt regarding patient sensitivity.

Storage and handling

Always treat the device carefully to avoid surface damage or alterations to the surface geometry. Care should be taken when handling this device. If surgical gloves are cut during handling, the risk of infection must be taken into account. Product should be handled with care as it may have sharp edges. Stick injury can cause damage to user and cross contamination of the device. Any sterile packed device that has flawed packaging, or has been contaminated after removal from the packaging, or is in packaging for which the shelf life has expired, should not be used.

